

BALMORAL II ACTIVE MATTRESS



Cleaning Guidance:

Cover should be wiped down with hot water and mild detergent. If heavily soiled, the cover may be machine washed up to 60-95°C and tumble dried. For disinfection use sodium hypochlorite solution max 1 000ppm. Use no bleach, biological or phenolic-based cleaners. Dry completely before storing.

Warranty

In terms of Harvest Healthcare Guarantee and Warranties the Harvest Balmoral Plus mattress is guaranteed for three years from date of purchase.

Exclusions:

• Damage to fabric covers through incorrect use and penetration by sharp instruments is excluded.

Supplier Contact Details:

t: +44(0)1709 377172 e: sales@harvesthealthcare.co.uk w: www.harvesthealthcare.co.uk Sheaf house, Bradmarsh Way, Bradmarsh Business Park, Rotherham UK, S60



Balmoral II cells and foam fitted underneath the alternating cells

Important Safety Notice

Harvest Heathcare Ltd supplesequipment and applances to cover a range of medeal condition. Speefic choice of equipment is the sole responsibility of the clinear protection real; er overseeing individual patient care. Harvest Heetheare Ltd supples a range of products such as foam mattresses, active air mattresses, tai protection mats and bed rai proteClon bumpers. These products are usual ycovered in vapour permeabe PU fabric that could cause rSkof suffocation in the event of proonged tacla contact. Mattress rSkcategoresare to be used as agudeony. RSk assessments specific to user must be carried out prior to the use of a products. PrexluCls and da18 cont ,..., a y undergo development and may therefore deviate from the details given. Version 1: June 2024